

Instructions for use - STERILUB®

INDICATIONS: STERILUB® is a single dose of sterile oil for single use and a universal adapter tip to lubricate after sterilisation the handpieces and contra angles for dentists and stomatologists. Delivered in double sterile packaging.



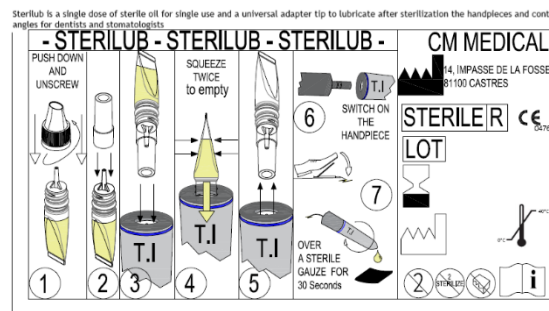
PRECAUTIONS FOR USE:

- Read the instructions carefully before using.
- Sterile medical device
- Single use medical device
- Dispose of the medical device after use
- Do not re sterilize
- Do not use after the expiry date
- Do not use if the packaging is damaged
- Use all the oil contained in the single dose
- Do not reuse STERILUB® due to the following risk:
 - Microbiological contamination of the handpiece
 - Insufficient lubrication for a second handpiece
- Do not use single dose or adapter tip if they are dropped on the floor
- Do not mix with another oil
- Immediately clean any oil outside of the handpiece or contra angle
- Do not use single dose without adapter tip
- Do not use on a medical device unsuitable for the adapter tip
- Store in a clean and dry place, protected from light.

- Handle with gloves and mask.
- Store between 0 and 40°C maximum


HOW TO USE STERILUB®

- Take the single dose out of the sterile package just before use.
 - Leave the adapter tip in the sterile package.
- 1 Hold the single dose cap up. Push down the security cap and unscrew.
 - 2 Insert the widest side of the universal adapter onto the tip of the single dose
 - 3 Insert the narrow end of the universal tip into the bottom of the handpiece (T.I) as far as it will go (the handpieces which are used with E-TYPE micromotors)
 - 4 Turn the handpiece assembly down, and STERILUB in upright position and squeeze the single dose twice to empty it completely
 - 5 Remove the single dose and the universal adapter from the T.I
 - 6 Switch on the handpiece
 - 7 Hold the handpiece over a sterile gauze for 30 seconds in order to eliminate any excess oil.
- Dispose of the single dose and universal adapter for recycling.













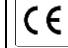
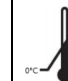
DEVICE PERFORMANCE:

- Post sterilisation lubrication
- Exact dose of oil in each single dose according to the requirements of the handpiece and contra angle of surgery manufacturers
- Sterile oil in accordance with the requirements of the European pharmacopoeia and without risk for patients as well as medical staff
- Sterile oil ready for use directly in the operating theatre
- Universal tip adapter for all surgical handpieces and contra angles which are used with E-TYPE micromotors

 CM MEDICAL 14, impasse de la fosse
81100 CASTRES – France

Date of the publication of the notice: 10 December 2018

 0476

	CM MEDICAL SAS 14, impasse de la fosse 81100 CASTRES		Disposable device – not for reuse
	Gamma ray sterilisation		Do not Re sterilize
	Batch number		Do not use if packaging is damaged
	Expiry date		Speciale instruction for use
	Date of manufacture		CE marking issued on
	Temperature of conservation		