



CM Médical was founded in 2015 by Christophe Micheau, who holds a university degree in hospital and industrial sterilisation and has been active in the dental industry since 1985. Since its creation, CM Médical has been closely linked to Gamasonic, the market leader in ultrasonic tanks and washer-disinfectors.

CM Médical is the exclusive distributor of Gamasonic in western France, Spain and Portugal.

During his thirty-five years spent in the medical sector, Christophe Micheau has observed the bad practices linked to the sterilisation and lubrication of surgical handpieces. He therefore decided to create STERILUB® in order to comply with good sterilisation practices in dental offices and hospitals.



© Castres Mazamet Technopole

## CONTACT

✉ [contact@sterilub.com](mailto:contact@sterilub.com)

☎ +33 7 85 57 77 94  
+33 5 63 72 04 64

For more info visit:

🔗 <https://sterilub.com/>



# STERILUB®

par CM MEDICAL

A revolutionary lubrication system  
that preserves the sterility of  
surgical instruments



## STERILISATION RULES FINALLY RESPECTED

Everybody knows that only clean and dry tools can be sterilised. But then why lubricate stomatology and dentistry hand tools before sterilization?

With STERILUB® only what is clean and dry will be sterilised, because the sterile lubrication of surgical handpieces and contra-angles is carried out in the OR just before surgery.

## PRESERVED STERILE BAGS

The various vacuum phases of the autoclave suck up almost all of the oil contained in the surgical handpieces and contra-angles. This can be seen inside the sterile bags that are stained after autoclaving.


Thanks to STERILUB® the sterile bags of handpieces and contra-angle handpieces are no longer stained.




## A SINGLE DOSE OF STERILE LUBRICANT



Insert the  
adaptor end cap

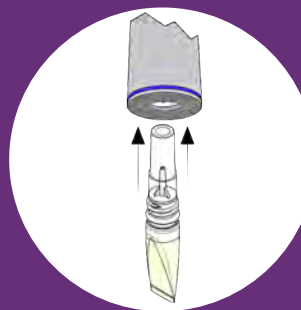

Insert both into the back  
of the handpiece

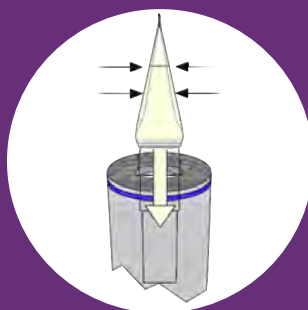

Press twice to empty  
the single dose...  
And it is ready to use!

## THE WELL-BEING OF THE PATIENT AND THE PRACTITIONER

The rise in temperature of the oil in contact with water vapour creates an emulsion which solidifies during the drying phase and prevents the proper lubrication of the instruments.

STERILUB® is more practical and safe for both the patient and the practitioner. Indeed, the risks of overheating and breakage disappear as lubrication is done directly in the OR in a totally aseptic way.

## EXTENDED LIFE OF HAND TOOLS

Frequent repairs of surgical handpieces and contra-angles are very expensive.

STERILUB® allows very significant savings on maintenance costs and extends the service life of surgical handpieces and contra-angles.

Universal end cap  
Monodose made with 50%  
recycled material, 100%  
recyclable  
Patent filed internationally  
DM Class 1S marking 