TECHNICAL  
SPECIFICATIONS SHEET  
  
**MEDICAL DEVICE**

*Review : 1.0*

*Date : 02/10/20*

**1. Administrative information about the company**

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**2.3 Definition :**

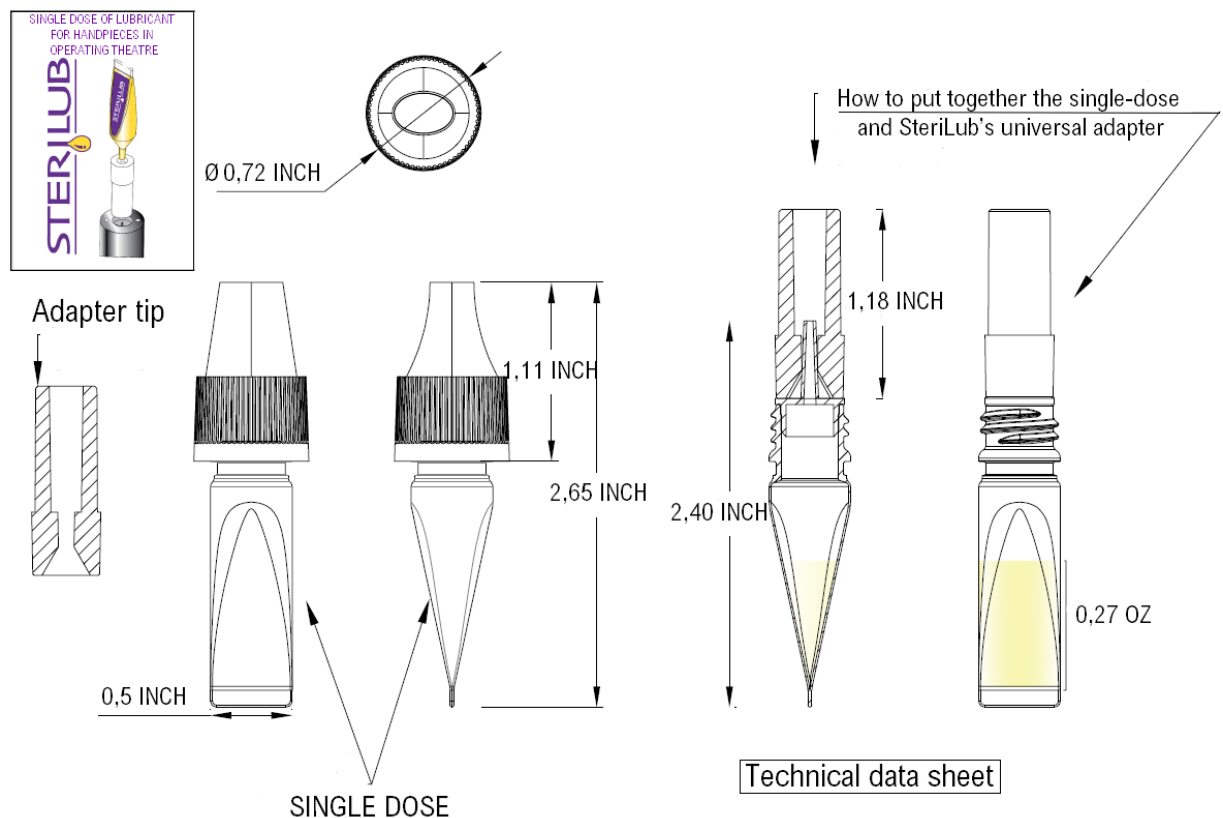
STERILUB® is a single dose of sterile single-use oil with its universal adapter for post-sterilisation lubrication of handpieces and contra angles in dentistry and stomatology.

## 2.4 Description of the device (including its photo, scheme, dimensions, volume, etc.) :

### 2.4.1 Product components

On the one hand, there is a single dose made of polyethylene (MARLEX HHM 5502BN) manufactured by blow moulding and containing 0.8 ml (0.028oz) of TURMOSYNTH VG 32 oil in accordance with the requirements of the European Pharmacopoeia. It is sealed with a safety screw cap which once unscrewed, shows a straight adapter tip.

On the other hand, there is a universal adapter tip made in polyethylene (PE 48 shore D) manufactured by injection moulding whose largest tip can be fixed onto the single-dose adapter tip.

### 2.4.2 Packaging

In order to ensure the sterility of the product, the monodose and the universal adapter are grouped, without being assembled, in the first paper/plastic package with a how-to-use sticker on it. They are then packed in the second container that is the same quality as the first one but a larger size with a how-to-use scheme sticker on it. The dimensions are as follows:

- The first package is 150 x 100 mm (5.9 x 3.93 inches)
- The second package is 250 x 150 mm (9.84 x 5.9 inches)

The packages are made of paper type NR 60 GSM when it comes to the paper side and polyester type 48 PET/200 LDPE when it comes to the plastic part.

<p><b>2.6</b></p>	<p><b><u>DM classification</u></b> : I Sterile  <b><u>Applicable EU Directive</u></b> : 93/42/CEE  <b><u>According to Appendix No. VII</u></b>  <b>Notified body and number</b> : Kiwa Cermet Italia (0476)   <b>Date of first placing on the market in the EU</b> : waiting for CE label</p>
<p><b>2.7</b></p>	<p><b>Registration according to the nomenclature of the public procurement code :</b></p>
<p><b>2.8</b></p>	<p><b>Conditioning / Packaging :</b>  <b>UCD</b> : (Unité de commande) box of 100 single-doses  <b>CDT</b> : (Multiple de l'UCD)  <b>QML</b> : (Quantité minimale de livraison) 1 box of 100 single-doses with the dimensions 600 x 400 x 200 mm, for a total weight of 1.725 kg</p>
<p><b>2.9</b></p>	<p><b>Labelling:</b> copy (facsimile) of the labelling template : see attached labelling  <b>UCD</b>  <b>CDT</b>  <b>QML</b>  <b>Barcode</b></p>
<p><b>2.10</b></p>	<p><b>Devices and accessories associated with listing.</b>  Universal adapter tip delivered with the single-dose</p>
<p><b>2.11</b></p>	<p><b>Composition of the device :</b>  Main components:</p>

Component	Ingredient	CAS Number	Functions	Concentration by %	Per single dose <sup>1</sup> (unitary formula)	Percentage (m/m)
Turmosynth VG 32 : White oil	<b>Turmosynth VG 32 :</b>	/	<b>Lubricant</b>	<b>100 %</b>	<b>6.88g</b>	<b>100</b>
	White mineral oil	8042-475	Base oil	99 %		
	Triphenyl phosphorothionate	597-82-0	Extreme / Pressure Antiwear additive	≤ 0,5 %		
	Hexamethylene bis[3-(3,5-di-tert-butyl-4-hydroxyphenyl)propionate]	35074-77-2	Antioxidant	≤ 0,2 %		
	Benzenamine, N-phenyl-, reaction product with 2,4,4 trimethylpentene	68411-46-1	Antioxidant	≤ 0,1 %		
	Amines, C11-14-branched alkyl, monoheptyl and dihexyl phosphates	80939-62-4	Corrosion inhibitor / Antiwear additive	≤ 0,1 %		
	N,N-bis(2-ethylhexyl)-4-methyl-1H-benzotriazole-1-methylamine	80584-90-3	Metal desactivator	≤ 0,1 %		
	N,N-bis(2-ethylhexyl)-5-methyl-1H-benzotriazole-1-methylamine	80595-74-0	Metal desactivator	≤ 0,1 %		
Anti foaming agent	/	Defoaming agent	≤ 0,05 %			
<b>Primary packaging</b>	<b>Packaging Elements</b>	<b>Raw Materials</b>	<b>CAS number</b>	<b>Characteristics</b>	<b>Functions</b>	<b>Reference to standard</b>
	Single dose	ALCUDIA® 5502E high-density polyethylene copolymer of hexene	25213-02-9	Good environmental stress cracking resistance	Container of lubricating oil	Complies with : - US Food and Drug Administration(FDA) 21 Code of Federal Regulation (CFR) 177.1520 - Regulation (EC) 1935/2004 - Regulation (EC) 2082/2008
	Pipette	Formolene® HB5202B	25213-02-9	excellent stiffness and stress crack resistance properties	Immediate packaging	- U.S FDA 21 CFR 177.1520
	Safety cap	MAXITHEN HP 1279 Linear low- density polyethylene	13463-67-7	Excellent thermal stability	Safety cap	- Food compliant
<b>Accessory</b>	Universal tip	LDPE 780E Low Density Polyethylene Resin	9002-88-4	Excellent flow Good rigidity Good surface gloss	Inserting of the single dose in the handpiece	Complies with : - Canadian Health products and Food Branch (HPFB) No Objection - US FDA 21 CFR 177.1520

<b>3. Sterilisation process :</b>	
	Delivered in double sterile package
	Gamma ray sterilisation

<b>4. Conditions of conservation and storage</b>	
	<b>Normal preservation &amp; storage conditions :</b> at room temperature, between 0 and 40°C, in a dry and dust-free place.
	<b>Special precautions :</b> do not exceed the expiry date
	<b>Duration of the validity of the product :</b> 5 years
	<b>Presence of temperature indicators if applicable :</b> NA

<b>5. Safety in use</b>	
<b>5.1</b>	<b>Technical security :</b> NA
<b>5.2</b>	<b>Microbiological safety (if applicable) :</b> NA (to complete) <b>Registration number :</b> NA

<b>6. Advice for use</b>	
<b>6.1</b>	<b>Instructions for use :</b> <ol style="list-style-type: none"> <li>1. Take the single dose out of the sterile package just before use.</li> <li>2. Leave the adapter tip in the sterile package.</li> <li>3. Hold the single dose cap up. Push down the security cap and unscrew.</li> <li>4. Insert the widest side of the universal adapter onto the tip of the single dose.</li> <li>5. Insert the narrow end of the universal tip into the bottom of the handpiece (T.I) as far as it will go (the handpieces which are used with E-TYPE micromotors)</li> <li>6. Turn the handpiece assembly down, and STERILUB in upright position and squeeze the single dose twice to empty it completely</li> <li>7. Remove the single dose and the universal adapter from the T.I</li> <li>8. Switch on the handpiece</li> <li>9. Hold the handpiece over a sterile gauze for 30 seconds in order to eliminate any excess oil.</li> <li>10. Dispose of the single dose and universal adapter for recycling.</li> </ol>
<b>6.2</b>	<b>Indications :</b> (CE norm)  No contraindications to the use of Sterilub®, nor undesirable effects during the use on patients
<b>6.3</b>	<b>Precautions of use :</b> <ul style="list-style-type: none"> <li>- Read the instructions carefully before using</li> <li>- Sterile medical device</li> <li>- Single use medical device</li> <li>- Dispose of the medical device after use</li> <li>- Do not resterilise</li> <li>- Do not use after the expiry date</li> <li>- Do not use if the packaging is damaged</li> <li>- Use all the oil contained in the single dose</li> <li>- Do not reuse STERILUB® due to the following risk:</li> </ul>

	<ul style="list-style-type: none"> <li>- Microbiological contamination of the handpiece</li> <li>- Insufficient lubrication for a second handpiece</li> <li>- Do not use single dose or adapter tip if they are dropped on the floor</li> <li>- Do not mix with another oil</li> <li>- Immediately clean any oil outside of the handpiece or contra angle</li> <li>- Do not use single dose without adapter tip</li> <li>- Do not use on a medical device unsuitable for the adapter tip</li> <li>- Store in a clean and dry place, protected from light.</li> <li>- Handle with gloves and mask</li> <li>- Store between 0°C (32°F) and 40°C (104°F)</li> </ul>
<b>7.</b>	<b>Additional information on the product</b>
	<ul style="list-style-type: none"> <li>- Contains the exact dose of oil (0,8ml) in each monodose according to the requirements of the manufacturers of handpieces and contra-angle handpieces for dentists and stomatologists.</li> <li>- Universal adapter tip for all handpieces and contra-angle handpieces that are used on a TYPE-E micromotor.</li> </ul>
<b>8.</b>	<b>List of appendices to the file</b>
	<ul style="list-style-type: none"> <li>CE certificate of conformity</li> <li>Labelling</li> <li>Instruction manual</li> <li>Brochure</li> </ul>

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Sterile Oil to lubricate dental and stomatologist handpieces and contra angles

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**Chief Operating Officer**  
**Giampiero Belcredi**

Digitally signed by: BELCREDI GIAMPIERO  
Date: 25/02/2021 09:06:10

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**CERMET**

Chief Operating Officer  
*Giampiero Belcredi*

Digitally signed by: BELCREDI GIAMPIERO  
Date: 25/02/2021 09:06:59

**CE**

Organismo Notificato n. 0476  
Notified Body nr. 0476

## **EC Declaration of Conformity**

According to Annex II of Directive 93/42/EEC on medical devices

We,  
CM Medical  
14 Impasse de la Fosse  
81100 – CASTRES – France

Declare under our sole responsibility that the medical device

### **STERILUB**

Classification : STERILUB® is a class Is medical device (in accordance with rule 1 of the annex IX of European Council Directive 93/42/EEC)

Complies with all the applicable requirements of Directive 93/42/EEC

Name and identification of the notified body : Kiwa Cermet Italia (Notified Body number : 0476) with the address of the headquarters : Via Cadriano, 23, 40057 Granarolo dell'Emilia BO,

This, for the period of validity of the certificate, i.e. until: 26/05/2024

Done at Castres on 26/02/2020

Christophe MICHEAU, CEO

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di Kiwa Italia Holding S.r.l.